

MAINS ELECTRIC MOBILE CHANGING BENCH OPERATING INSTRUCTIONS

IMPORTANT!

**PLEASE TAKE TIME TO FAMILIARISE YOURSELF WITH THE USE
AND MAINTENANCE OF THIS EQUIPMENT
MAKE SURE THAT CHILDREN ARE KEPT WELL OUT OF THE WAY
AND THAT NO ONE IS LIKELY TO INTERFERE OR TOUCH THE
BENCH WHILST IN USE**

TO RAISE

1. SWITCH UNIT ON AT THE ISOLATOR SWITCH
2. RAISE OR LOWER TO SUITABLE HEIGHT FOR TRANSFER USING THE HAND CONTROL
3. TRANSFER PATIENT ONTO THE CHANGING BENCH
4. RAISE SAFETY RAIL(S)!- ENSURING THEY ARE CORRECTLY ENGAGED
5. RAISE BENCH TO SUITABLE HEIGHT FOR OPERATOR

TO LOWER

1. LOWER TO SUITABLE HEIGHT FOR TRANSFER
2. LIFT SAFETY RAILS SLIGHTLY, SWING THEM DOWN TO HORIZONTAL AND PUSH THEM UNDER THE BENCH
3. TRANSFER PATIENT TO CHAIR , WHEEL CHAIR OR INTO STANDING AS CONVENIENT

HEALTH AND SAFETY

- **NEVER LEAVE A PATIENT UNATTENDED**
- ALWAYS LOCK ALL FOUR CASTORS PRIOR TO OPERATION
- CLEAN AND DRY UPHOLSTERY AFTER USE
- DO NOT ALLOW ANYONE TO PLAY WITH THE BENCH
- ALWAYS USE THE SAFETY RAIL(S)
- CHECK OPERATION OF THE SAFETY RAIL(S) BEFORE USE
- DO NOT EXCEED MORE THAN SIX COMPLETE CYCLES PER HOUR AT MAXIMUM RATED LOAD
- **NEVER EXCEED THE MAXIMUM LOADING OF 200KG**

MAINTENANCE

- CHECK WEEKLY THAT ALL VISIBLE SCREWS AND FITTINGS ARE IN PLACE
- SET UP A ROUTINE MAINTENANCE SCHEDULE AS RECOMMENDED IN THE PRODUCT INFORMATION FOR USE BOOKLET.

